

Instruction for use of the measuring spoon (art. no. 1009xx)



Manufacturer of the measuring spoon:

Manfred Schägner GmbH
Industriestr. 3
76479 Steinmauern
Germany
phone: +49 7222 9274 0
FAX: + 49 72 22 690 36
e-mail: info@schaegner.de
www.schaegner.de

Please note that following wording of the instruction for use is a suggestion. As Schägner as legal manufacturer of the measuring spoon has no specific information about the medicine the measuring spoon is packed with, it is up to the pharmaceutical company that encloses the measuring spoon to the package that contains the medicinal product to adapt the instruction for use according to the specific requirements of the pharmaceutical company's medicinal product (see also "Demarcation of responsibilities" as stipulated by contract between Schägner and pharmaceutical company).

Suggestion for the instruction for use of the measuring spoon:

"The measuring spoon is intended to be used for the dosage and oral use of liquid medicinal products.

For this purpose the measuring spoon is placed on a flat surface. The liquid to be dosed is filled into the spoon carefully until the liquid level reaches the measuring line $\frac{1}{2}$ (=2,6 ml) resp. $\frac{1}{1}$ (=5,2 ml) according to the described dosage in the leaflet of the medicinal product.

Subsequently the filled spoon is introduced carefully into the mouth by holding the spoon grip and the liquid can be swallowed.

The measuring spoon is designed by the manufacturer for multiple use limited to the exhaustion of the medicinal product the spoon is marketed with and should be disposed of as household waste after use. After each use the spoon has to be rinsed with water containing dishwashing detergent and wiped with a paper towel.

Note on safety:

- measuring spoons that show obvious damages or illegible graduation marks should not be used but returned to the pharmacy, pharmaceutical company or to the manufacturer"

Dr. Ute Linder
Manfred Schägner GmbH
Safety manager medical devices

